

Dura-Band Wristbands

Application Instructions

The Dura-Band identification wristband is designed for comfort and ease of use. The unique adhesive fitting and closure system makes it easy to fit the Dura-Band to any wrist size.



STEP 1
 Remove backing
 from fitting tab (1)



STEP 2
 Place the Dura-Band
 around wrist



STEP 3
 Affix the fitting
 tab (1)



STEP 4
 Remove backing
 from closure tab (2)



STEP 5
 Affix closure
 tab (2)



STEP 1: Remove the backing from the fitting tab (1)

Before applying to patient's wrist, first remove the backing from the small fitting tab (1) on the back of the Dura-Band. Discard the backing.

STEP 2: Place the Dura-Band around the wrist

Place the Dura-Band around the wrist, with the fitting tab (1) at the bottom, facing towards you.

STEP 3: Affix the fitting tab (1)

Bring the bottom of the Dura-Band up around the wrist, and affix the fitting tab to the underside of the Dura-Band at a location where the Dura-Band fits correctly and comfortably around the wrist.

STEP 4: Remove the backing from the closure tab (2)

Remove the backing from the larger closure tab (2) on the back of the Dura-Band. Discard the backing.

STEP 5: Affix the closure tab (2)

Wrap the top of the Dura-Band around the wrist, and affix the closure tab (2) to the outside of the Dura-Band. For user comfort, make sure the closure tab is centered on the surface of the Dura-Band with no exposed adhesive on either side. Press on the closure tab to assure positive adhesion.

After the closure tab has been affixed, do not attempt to re-open it. The closure tab is equipped with tamper-evident slits that are designed to tear if the wristband or closure has been opened or tampered with.

Part Numbers:

819011 Single roll - 125 wristbands

819012 Carton - 500 wristbands